**TETANOS QUICK STICK  
CONTROL SET**

**Reference :  
TE2C / S-GZ-CZ**

**ENGLISH (en)**

**FRANÇAIS (fr)**

**EUROBIO SCIENTIFIC**

7 avenue de Scandinavie

ZA de Courtaboeuf

91940 Les Ulis – France

Tel : +33 1 69 79 64 80 / Fax : 01 69 79 05 35

[www.eurobio-scientific.com](http://www.eurobio-scientific.com)

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Version tracking:

The areas highlighted enable to identify words, lines or paragraphs changed since the previous version

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## TETANOS QUICK STICK: CONTROLS SET

Reference: TE 2C / S-GZ-CZ

For Research Use Only  
(Professional use only)

### 1. INTENDED USE

The set consists of two human sera : a negative control and a positive control for antibodies against Tetanus Toxoid. Those controls have to be used with the Tetanos Quick Stick (Ref. S-GZ-1, TE20B / S-GZ-20B, TE40B / S-GZ-40B R) as quality controls during routine test or training.

### 2. SET CONTENTS

Negative control : 1 dropper of 1 ml  
Positive control : 1 dropper of 1 ml  
Human Serum + NaN<sub>3</sub> 0.05 %

### 3. PRECAUTIONS

1. *In vitro* use only.
2. The sera have been tested for HbS antigen, anti HIV and anti HCV antibodies and have been found non reactive. Handle all the serum as if they contain infectious agents.
3. Wear protective clothing such as laboratory coats and disposable gloves while assaying serum.
4. Do not eat, drink or smoke in the area where specimens and kit reagents are handled.
5. Avoid any contact between hands and eyes or nose during serum collection and testing.
6. The serum contain 0.05 % sodium azide. Users should be aware of its toxic properties if absorbed or ingested.

### 4. STORAGE AND STABILITY

Store the control set at **2 – 8°C**.  
The controls can be used until (see expiry date indicated on the dropper).  
Do not freeze the serum controls.

### 5. TEST PROCEDURE

1. Remove two test units (Tetanos Quick Stick) from the protective wrapper (by tearing along the split) and place on a level surface.
2. By holding the dropper vertically dispense **one drop** of each control into the sample well.
3. Add **3 drops of diluent** immediately, holding the flask vertically above the sample well and avoiding touching it with the dropper.
4. The test should be read **10 minutes** after adding the diluent to the sample well.  
NB: If no colored liquid appears in the second window after 2 minutes, one more diluent drop can be added in the sample well.

### 6. INTERPRETATION OF RESULTS

#### a) Test validation

One pink colored line appears in the control window ("C") showing that the test has been carried out correctly.  
If no line appears in the control window, the test should be repeat with another test unit.

#### b) Interpretation

- No apparent colored line in the test window ("T"): **the test is negative**.
- In addition to the colored line in control window ("C"), a clearly distinguishable pink colored line also appears in the window ("T"), indicating a **positive result**. (The differences of intensity between the lines does not affect the interpretation of the results)

#### Expected values:

- The negative control should be negative : no apparent colored line in the test window ("T").
- The positive control should be positive : apparent colored line in the test window ("T").

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